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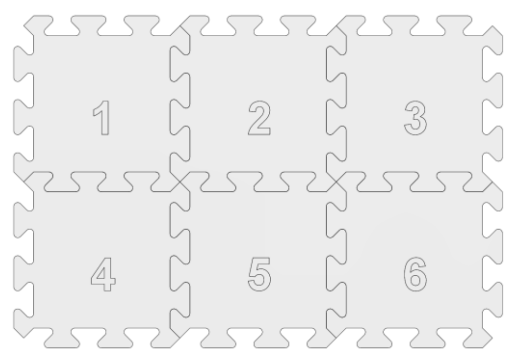
# ENG - INSTRUCTIONS FOR USE

Dear User,

Please read the instructions for use carefully, which contain important information on the use of the WellStepp® Kids medical device.

The WellStepp® Kids sensory development mat is recommended by orthopaedic specialists and physiotherapists as a developmental tool for young children.

## Medical technical description

The WellStepp® Kids sensory development mat is a tool for improving daily life by modelling the natural environment. It is used to artificially create an unstable state of the human body, stimulating the deep muscle and articular sensations to stabilise the body's momentary unstable state through subconscious movements of muscle tension.

## Effects of the WellStepp® Kids sensory development mat

- it strengthens the musculature of the foot arch
- it helps prevent developing flat feet
- it improves posture
- it improves the sense of balance
- it supports healthy neurodevelopment it improves perception it develops fine motor movements

## Description of the product

The WellStepp® Kids sensory development mat, with its shapes resembling a shingle beach and its pleasant surface, turns the strengthening of children's foot arch musculature into a fun game. Its special design provides a comfortable and effective solution for preventing possibly lifelong orthopaedic disorders that may develop in childhood. With regular use, it can prevent and treat flat feet and other complaints, help maintain a healthy posture, and support proper neurodevelopment.

The WellStepp® Kids sensory development mat can be used both at home and in institutional settings. Its use does not require any special expertise, but we recommend following the suggestions below to achieve the best results.

Although the word 'sensory' is included in the name of the product, there is no sensor in the product itself; the term refers to the fact that the product is designed to be used for doing sensory exercises.

The product is not a toy for children; it is specifically designed for gymnastic development activities.

For the performance of the product, the description of the product should be interpreted in the context of the instructions for use as a whole.

## Recommended exercises for the WellStepp® Kids sensory development mat

You can scan the QR code below using your phone or tablet and click on the displayed link to open our website.

On our website, you will find a series of fun exercises developed by physiotherapists for the effective use of the WellStepp® Kids sensory development mat.

Our website: [www.wellstepp.eu](http://www.wellstepp.eu)

## Instructions to follow for safe and effective use

The product should always be used under parental supervision.

The sensory development mat is designed for indoor use.

Outdoor environmental conditions may affect long-term properties in terms of quality and aesthetics. For optimum results and safe use, the product is recommended to be used on even ground, preferably on surfaces that are as non-slippery as possible.

The product does not contain any substance harmful to health, but children could bite off small pieces, which in extreme cases could lead to choking. Make sure that the child/children do not put the mat in their mouths or lick it.

Recommended: From the age of 1

## Contraindications

The use of the WellStepp® Kids sensory development mat is not recommended if, in the opinion of a specialist/physiotherapist, the child has musculoskeletal complaints where health development using artificial instability combined with independent movement is not recommended or not feasible.

WellStepp carpet size: 58 x 154 cm

# WellStepp

## sensory development mat kids

## Necessity to consult a healthcare professional

If it is not possible to view the exercises made available on the internet or if there is a problem with understanding the exercises, it is recommended to consult a healthcare professional (orthopaedic physician, physiotherapist) to ensure that the correct exercises are performed.

## Cleaning and storage

The product can be easily cleaned with a damp cloth; wait for the mat to dry completely before use. If used daily, wipe both sides of the product with a sponge every two weeks with water mixed with mild detergent/shampoo/fabric softener, then leave to dry. Store at normal room temperature, folded to shipping size.

It is recommended to store the mat in its original box if possible. Although the material is durable, it may deform under prolonged force, which may reduce its functional value.

## Technical information

Carpet Size: 58 x 154 cm

Puzzle Size: 50x73,5 cm

Material: 100% polyethylene foam

The product is not suitable for human consumption. When used as intended, it does not pose a health hazard, and it does not contain any substance harmful to health.

## Manufactured by:

Polifoam Co. Ltd.

H-1097 Budapest, Táblás u. 34/A.

Place of origin: EU

Date of issue / latest update of the instructions for use: April 2025

## Meaning of the symbols on the packaging

The product is manufactured using 100% renewable electricity.

This symbol indicates the serial number of the product.

This symbol reminds you to read the instructions for use.

This label certifies that the product made available on the market has been tested by the manufacturer and complies with EU safety, health, and environmental standards.

## Manufacturer's Declaration

Polifoam Co. Ltd. (H-1097 Budapest, Táblás u. 34/A.) hereby declares that it issues this declaration as a manufacturer under its sole responsibility in accordance with Annex IV of Regulation (EU) 2017/745 of the European Parliament and of the Council of 05.04.2017 on medical devices (hereinafter referred to as "Regulation").

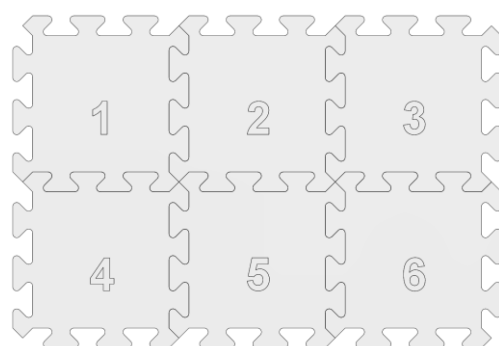
Polifoam Co. Ltd. as the manufacturer declares that the WellStepp® Kids sensory development mat for children is a Risk Class I medical device. The Company has fulfilled its obligation for registration in respect of the product and as an economic operator as well.

Polifoam Co. Ltd. further declares that the device manufactured by it complies with the general safety and performance requirements, the requirements regarding design and manufacture, as well as the requirements regarding the information supplied with the device as set out in Annex I to the Regulation, and that it has the Technical Documentation as set out in Annex II to the Regulation as well as the Technical Documentation on Post-market Surveillance as set out in Annex III to the Regulation.

After the delivery of the product, following the occurrence of a serious incident related to the use of the device during the period of use, Polifoam Co. Ltd. as the manufacturer of the device has an obligation to report to the National Institute of Pharmacy and Nutrition (OGYÉI, email: [amd.vig@ogyei.gov.hu](mailto:amd.vig@ogyei.gov.hu)).

If you have purchased a WellStepp puzzle product, it is a good idea to store the batteries disassembled. The correct assembly of the puzzle product is indicated by the numbers on the back of the elements. For the best results, it is recommended to assemble the product according to these dimensions.

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